BOWMAN® Dispensers Organize-Contain-Comply

RS002–0212 Respiratory Hygiene Station – Locking



Related Products (each sold separately):

RS002-0512: Respiratory Hygiene Station - Locking, Aluminum (shown below)



Product Description:

- Respiratory Hygiene Station Locking
- Holds two boxes of face masks and one two boxes of facial tissues or one box of gloves
- Hinged locking lid
- Keyholes for wall mounting
- Quartz Beige ABS Plastic

Primary Area for Product Usage:

· Anywhere respiratory hygiene products are needed

Dispenses:

- Left: Box(es) no larger than 7.75"W x 4.00"H x 4.75"D (19.7 cm x 10.2 cm x 12.1 cm)
- Right: Box(es) no larger than 5.50"W x 10.00"H x 4.75"D (14.0 cm x 25.4 cm x 12.1 cm)


Package Specifications:

- Package Quantity: 1 per case
- 24"L x 16"W x 6"H (61.0 cm x 40.6 cm x 15.2 cm)
- 6.0 lbs (2.7 kg) approximated

Product Specifications (overall external dimensions):

- 13.80"W x 12.65"H x 5.30"D (35.1 cm x 32.1 cm x 13.5 cm)
- 2.0 lbs (0.9 kg) approximated

Mounting Specifications:

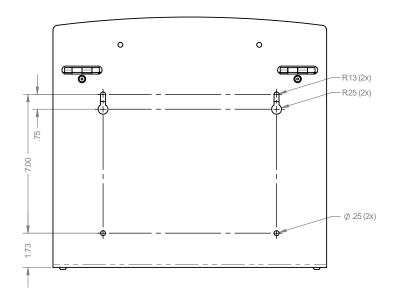


BOWMAN[®] Dispensers Organize-Contain-Comply

RS002–0212 Respiratory Hygiene Station – Locking


Installation Instructions/Characteristics & Care

Installation

Always mount dispensers securely prior to loading with product.

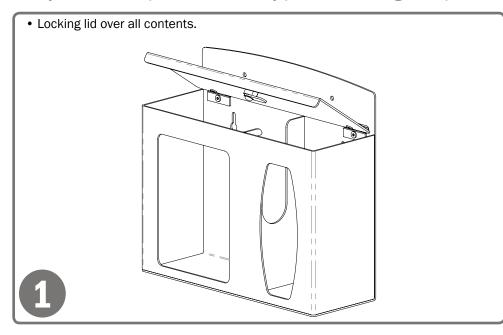

 Mount securely to a wall with screws approved and supplied by building maintenance.

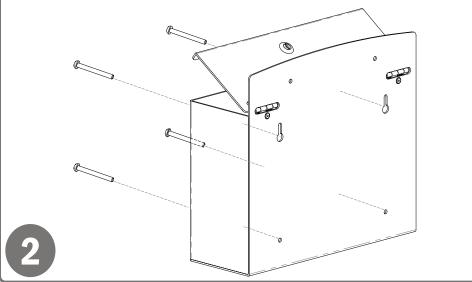

Parts included:

- Dispenser
- Tools Required:
 - No tools required to assemble dispenser

Care and Construction ABS Plastic

Material Characteristics

Designed for Long-Term, Heavy-Duty Applications

The combination of copolymers (Acrylonitrile, Butadiene, and Styrene) gives ABS an excellent combination of strength, rigidity, and toughness. The fauxwood finish is ARVINYL laminate created from a semi-rigid film.

ABS is chemical and stress-crack resistant. It is tough, dimensionally stable, and will hold up to long-term use. To ensure product longevity, fauxwood products should not be mounted in very hot or cold areas.

Care and Cleaning

Use mild soap and water to clean. Petroleum-based oils, solvents, and paints should not be used.

Standard hospital cleaning supplies are approved for use on our ABS plastic products.

Information subject to change without notice. Dimensions are approximate. (Not to scale) 800.962.4660 • www.BowmanDis ©2019 BOWMAN® Manufacturing Company, Inc. All Rights Reserved. Product specifications subject to change without notification. Consumables not included. R50020212TDSa

BOWMAN Dispensers 20325 71st Ave NE, Suite D, Arlington, WA 98223 800.962.4660 • www.BowmanDispensers.com